
STOPPI®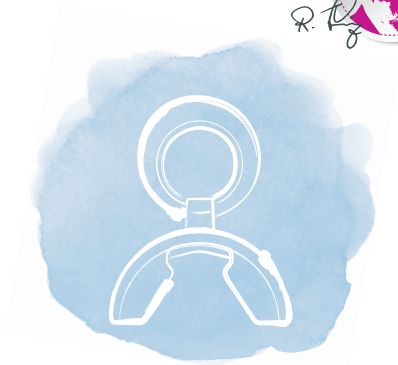

D
GB
F
I
E

Patienteninformation

STOPPI® zur Prophylaxe

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
Patient information

STOPPI® for prophylaxis

STOPPI® is used similarly to a normal pacifier. However, the silicone shield is placed between teeth and closed lips. Two small lateral bite planes for the molars provide the necessary support. **STOPPI®** is a healthy substitute to the damaging pacifier, thumb-sucking or baby bottles. **STOPPI®** is offered to children as substitute, which they voluntarily accept and wear because they are not forced to.

er.
eth
he
s a
hb-
en
ear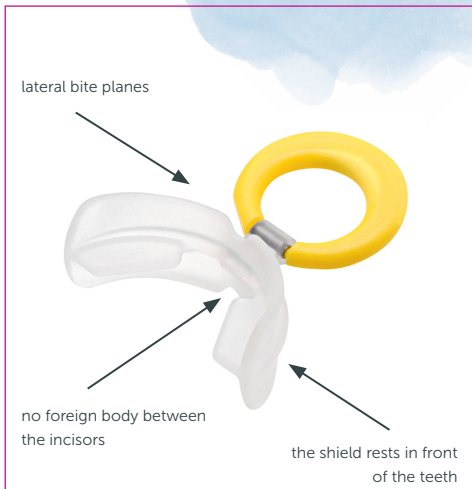

STOPPI®-rules

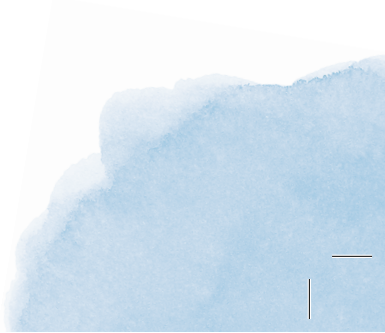
Your support and a few rules help your child:

- 1 At the age of two, **STOPPI®** is the only thing that should be worn in the mouth for extended periods. From this age, pacifiers, baby bottle and thumbs should be taboo.
- 2 **STOPPI®** is a substitute for thumb and dummy. The substitution must be voluntary.
- 3 **STOPPI®** should be used as a substitute for a pacifier, thumb or baby bottle and should only be worn at customary sucking times.

4

5

6


- 
- 
- 
- 4 As a rule, weaning is successful within a few weeks. Then the old detrimental sucking habits are forgotten.
 - 5 **STOPPI®** can be stored in the hygiene box
 - 6 Do not forget to brush teeth and **STOPPI®** regularly.

How is STOPPI® worn?

STOPPI® is used like a regular pacifier. But: it must be placed in front of the teeth – that means: between lips and teeth! Only the molars bite on the shield and provide the needed support!

- 1 Initially push one side of **STOPPI®** in front of the teeth,
- 2 then slightly press it and
- 3 now place the other side of **STOPPI®** between teeth and lips.

be
ips
o-
he
en

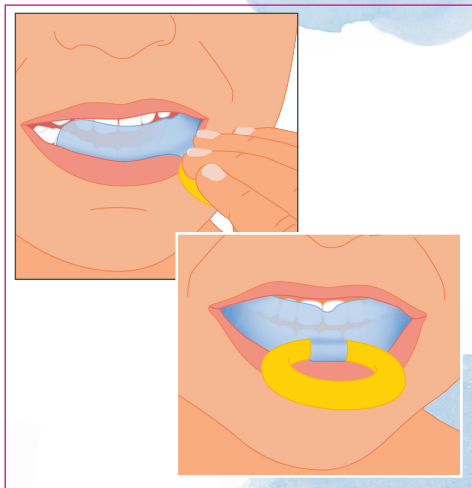CAUTION! For your child's safety:

Only use special ribbons which comply with EN 12586. Never attach **STOPPI®**, to other ribbons. Your child could strangle itself with them.

- Cracks, bite marks, stickiness or form changes could lead to detaching and swallowing of parts (risk of suffocation).
- Replace **STOPPI®** after one or two month of usage for reasons of hygiene and safety.

Th
un
ifie
ag
ST
ST
Ple
co

36.
ild
uld
of
for

The **STOPPI®**-Box is not a toy. It needs to be used only under adult supervision. **STOPPI®** is not a normal pacifier or soother! It is not suitable for children under the age of 24 months. Risk of suffocation! Only use **STOPPI®** under dental supervision. Do not use **STOPPI®** longer than recommended by the dentist. Please keep safe the instructions for use. In case of complaint: Keep LOT and AN

Instructions for use

Before using **STOPPI®** for the first time, clean it thoroughly under warm running water. Check **STOPPI®** regularly for places that have been bitten through! Replace **STOPPI®** when necessary. **STOPPI®** never dip in sweet food or medicine (causation of caries and/or material damages)!

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, you can also use the OROFAN® dental cleaning tablets. **STOPPI®** is neither dishwasher-safe nor microwave-safe. Only clean the hygienic **STOPPI®**-Box with a damp cloth.

CE Made in Germany **MD** Medical device

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