# OrthoP PreventAligner®

PR - M soft

D
GB
F
I
E

**Patienteninformation**

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D
<b>GB</b>
F
I
E

**Patient information**

## Prevention-oriented early treatment

Harmful sucking habits on the soother, thumb or other bodies and dysfunctions concerning the oral musculature, which are often associated with general postural weaknesses, permanently impair facial growth and tooth positions. They require early, i.e. **preventive, treatment**.

Habits and harmful dysfunctions such as

- open mouth posture
- incorrect tongue rest positions
- weak muscle lip closure and
- infantile swallowing (baby swallowing)

can be corrected with the **OrthoPreventAligner® PR-M**.

If malocclusions have already developed, they are encouraged to heal themselves - provided that the **OrthoPreventAligner®** is worn regularly.

## How jaw anomalies develop

Persistent **sucking habits** - of whatever kind - cause the tongue to be displaced backwards and downwards. It no longer rests against the teeth, thus causing an imbalance between the tongue and cheek muscles.

The upper jaw is constricted. There is a lack of space, which results in crooked incisors.

Even with an **open mouth posture**, the tongue only lies in the lower jaw and as such, it also causes the upper jaw to be too narrow.

When swallowing incorrectly, which also happens unconsciously about 1,500 times a day, the tongue always pushes itself between the incisors and shortens them.

The result: **An open bite.**

For the treatment of faulty swallowing and speech errors, logopaedic therapy should also be provided.

## The OrthoPreventAligner® PR-M

The **OrthoPreventAligner® PR-M** is made of highly elastic silicone. It has an ideal semi-circular shape for deciduous teeth, into which the teeth can be aligned by the restoring force of the silicone. In order to achieve healthier nasal breathing in the case of existing **mouth breathing**, the device deliberately closes the mouth without air slits.

More than half of all preschool children have an open mouth posture which, if left untreated, leads to a jaw anomaly.

Lip training with the **OrthoPreventAligner® PR-M** also eliminates this malfunction.

By shielding disturbing soft tissues in the mouth (tongue and cheek mucosa) and activating the muscles of the mouth and lips, teeth are able to move into the correct position by themselves.

**PR-M Primary dentition**

In the case of an open bite with prolonged vestibular pelotte (*Similar to Fig.*)

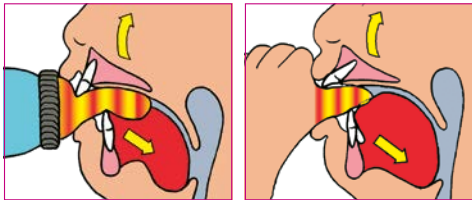
## About habits and malfunctions

**Habits** are understood as harmful sucking habits on fingers, thumbs, soothers and other foreign objects. They are referred to as **dynamic dysfunctions**. In order to help children break these harmful habits, the **OrthoPreventAligner® PR-M** can be worn as a preventive device and replacement for suckers or thumbs.

Incorrect swallowing - a habit from infancy - the wrong tongue rest position, an open mouth posture and mouth breathing are known as **static dysfunctions**.

All such conscious and unconscious habits are disruptive factors in jaw and facial development. They lead to "crooked teeth" and to misalignment of the jaws in relation to each other.

Wearing the **OrthoPreventAligner® PR-M** regularly can help to eliminate these disturbances and minor tooth misalignments that have already developed.

## Recommended wear

In order to break harmful habits and to eliminate **mal-functions** as well as to classify “crooked” teeth, the **OrthoPreventAligner®** should be worn regularly for at least 2 hours a day and at night.

## Wearing check

	Week 1		Week 2		Week 3	
	 hr.	 hr.	 hr.	 hr.	 hr.	 hr.
MON						
TUE						
WED						
THUR						
FRI						
SAT						
SUN						

## Instructions for use

Rinse the **OrthoPreventAligner®** with warm water before using for the first time. The **OrthoPreventAligner®** can be disinfected briefly by boiling water after infections.

## Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, OROFAN® dental cleaning tablets can be used.

The **OrthoPreventAligner®** is not dishwasher or microwave safe. Only clean the hygienic **OrthoPrevent Aligner®** box with a damp cloth.

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