OrthoP preventAligner®

PR - W hard

D

GB

F

I

E

Patienteninformation

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D
GB
F
I
E

Patient information

Prevention-oriented early treatment

Harmful sucking habits and dysfunctions of the oral musculature, which are often associated with general postural weaknesses, have a lasting effect on facial growth and tooth positions. They require **early, i.e. preventive, treatment**.

With the **OrthoPreventAligner®**-system, functional disorders such as open mouth posture, an incorrect tongue rest position and muscle-weak lip closure are corrected. If malocclusions have already developed, they are encouraged to heal themselves.

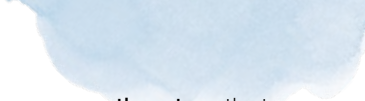
Existing swallowing by infants and all other harmful habits are also corrected and tooth misalignments that have already developed are reversed. If permanent incisors break through irregularly, the **OrthoPreventAligner® PR-W** guides them into the correct tooth position.

How jaw anomalies develop

Persistent sucking habits - of whatever kind - cause the tongue to be displaced backwards and downwards. It no longer lies behind and against the teeth of the upper jaw.

This causes an imbalance between the tongue and cheek muscles, narrowing the upper jaw. The resulting lack of space causes crooked incisors.

Similarly, with an **open mouth posture**, the tongue only lies in the lower jaw, resulting in a narrow upper jaw. When **swallowing incorrectly**, which happens unconsciously about 1,500 times a day, the tongue repeatedly pushes itself between the incisors, shortening the upper front teeth and causing an **open bite**.

For the treatment of faulty swallowing and speech errors, logopaedic therapy should also be provided.

The OrthoPreventAligner® PR-W

The **OrthoPreventAligner® PR-W** is made of highly elastic silicone.

Its effectiveness lies in the restoring force of the base material. It can align incorrectly positioned teeth into the ideal dental arch. However, these tooth movements are limited and cannot be compared to the possibilities of removable or fixed braces.

The shielding of disturbing soft tissues in the mouth (tongue and cheek mucosa) by the **Ortho Prevent Aligner® PR-W** causes crooked teeth to move into the correct position by themselves, while activating the mouth and lip muscles at the same time.

PR-W Mixed dentition

In the case of an open bite with
prolonged vestibular pelotte
(Similar to Fig.)

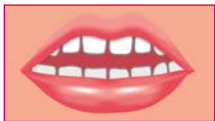
About habits and malfunctions

Habits are understood as harmful sucking habits on fingers, thumbs, soothers and other foreign objects. They are referred to as dynamic dysfunctions. In order to break the harmful habits, the **OrthoPreventAligner® PR-W** can be used as a preventive device and replacement for suckers or thumbs.

Incorrect swallowing - a habit from infancy - as well as the wrong tongue rest position, an open mouth posture and mouth breathing are known as static dysfunctions.

All such conscious and unconscious habits are disruptive factors in jaw and facial development. They lead to "crooked teeth" and to misalignment of the jaws in relation to each other.

By wearing the **OrthoPreventAligner® PR-W**, regularly, it helps to break these habits and to eliminate any minor tooth misalignments that have developed.

by sucking

by thumb sucking

Recommended wear

In order to break harmful habits and to eliminate **mal-functions** as well as to classify “crooked” teeth, the **OrthoPreventAligner®** should be worn regularly for at least 2 hours a day and at night.

Wearing check

	Week 1		Week 2		Week 3	
	 hr.	 hr.	 hr.	 hr.	 hr.	 hr.
MON						
TUE						
WED						
THUR						
FRI						
SAT						
SUN						

Instructions for use

Rinse the **OrthoPreventAligner®** with warm water before using for the first time. The **OrthoPreventAligner®** can be disinfected briefly by boiling water after infections.

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, OROFAN® dental cleaning tablets can be used.

The **OrthoPreventAligner®** is not dishwasher or microwave safe. Only clean the hygienic **OrthoPrevent Aligner®** box with a damp cloth.

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