OrthoP PreventAligner®

BR-W

D

GB

F

I

E

Patienteninformation

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E

Patient information

Prevention-oriented early treatment

Harmful factors have a lasting effect on facial growth, tooth malocclusions and the musculature of the jaw and mouth region. From experience, these have shown that they are transmitted to the permanent dentition, thus requiring the need for early **preventive, i.e. prophylactic, treatment**.

In addition to dysfunctions of the oral musculature and harmful sucking habits, **nocturnal teeth grinding** (bruxism) is also one of the harmful factors.

Because bruxism "**chews away**" and damages the enamel, it is also necessary to protect the tooth structure at an early stage.

Although about 20% of school children grind their teeth between the ages of 6 and 9, too little attention has been paid to bruxism in children and adolescents.

How bruxism occurs

The causes for nocturnal grinding (bruxism) have not yet been scientifically proven.

While it is assumed that stress triggers the increased chewing muscle activity in adults, the necessary **"grinding in"** of the milk teeth which are breaking through to each other is attributed to the grinding of small children (of whom approx. 15 to 20 % are affected).

So there is only conjecture, but no proof!

Even the consideration that in children and adolescents, the changed tooth positions are caused by orthodontic treatment has not been confirmed.

Prof. Hinz is convinced that the chewing movements that begin after the milk teeth break through are just as instinctive as the need to suck immediately after birth and that they continue to exist as habits.

OrthoPreventAligner® BR-W

Ortho PreventAligner® BR-W (bruxism) is made from a highly elastic silicone that resists the strong chewing pressure that arises from the night-time grinding of teeth.

Its effectiveness comes from the clenching of the teeth: The **Ortho PreventAligner® BR-W** prevents grinding and leads to the relief of the jaw joints. At the same time, the teeth are protected from abrasion and damage to the enamel. If there are other malfunctions caused by the tongue or incorrectly positioned teeth, these are also eliminated.

If the **Ortho PreventAligner® BR-W** is exclusively indicated due to bruxism, it is, in contrast to other versions of the device, only worn at night.

BR-W Transitional dentition
(Fig. similar)

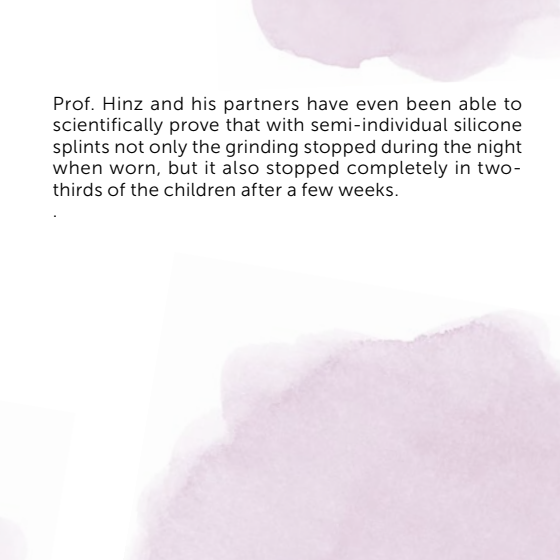
About bruxism

Nocturnal teeth grinding, which frequently occurs in small children, is considered by scientists to be one of the causes and risk factors for temporomandibular joint and masticatory muscle disorders (craniomandibular dysfunction = CMD) in adults. More than one million "**grinding splints**" are prescribed for them every year in Germany alone.

Until now, these splints have not been considered suitable for schoolchildren because new splints would be needed all the time due to the change of teeth.

This is not necessary with the prefabricated **Ortho PreventAligner® BR-W** which is also more economical than individually manufactured plastic splints.

Prof. Hinz and his partners have even been able to scientifically prove that with semi-individual silicone splints not only the grinding stopped during the night when worn, but it also stopped completely in two-thirds of the children after a few weeks.

Recommended wear

In order to break harmful habits and to eliminate **mal-functions** as well as to classify "crooked" teeth, the **OrthoPreventAligner®** should be worn regularly for at least 2 hours a day and at night.

Wearing check

	Week 1		Week 2		Week 3	
	 hr.	 hr.	 hr.	 hr.	 hr.	 hr.
MON						
TUE						
WED						
THUR						
FRI						
SAT						
SUN						

Instructions for use

Rinse the **OrthoPreventAligner**[®] with warm water before using for the first time. The **OrthoPreventAligner**[®] can be disinfected briefly by boiling water after infections.

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, OROFAN[®] dental cleaning tablets can be used.

The **OrthoPreventAligner**[®] is not dishwasher or microwave safe. Only clean the hygienic **OrthoPrevent Aligner**[®] box with a damp cloth.

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