# Friedensstifter®

1

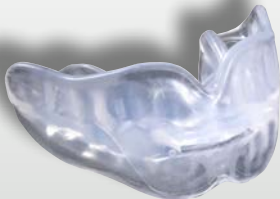
Abb. ähnlich

D
GB
F
I
E

## Patienteninformation

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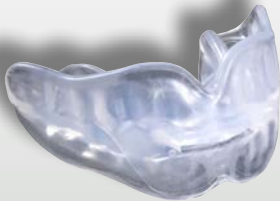
## Patient information

## **Wearing instructions**

During the first few days, in order to help you get used to it, wear the Friedensstifter® for about one hour before retiring to bed.

## **Instructions for use**

- Rinse the Friedensstifter® with cold water before using it for the first time.
- Check the Friedensstifter® regularly and watch out for any areas which are bitten through!
- Replace the Friedensstifter® if necessary.

## **Cleaning instructions**

The Friedensstifter® should be cleaned under running cold water – preferably with a toothbrush. Alternatively, OROFAN® cleaning tablets can be used for disinfection.

## **Snoring therapy system according to Prof. Hinz**

- Ready-made immediate measure against snoring
- Ideal entry-level system
- Optimal preparation for further complex treatment

## **Consequences and causes of snoring**

Snoring is a widespread problem which, although often considered harmless, it is always perceived as annoying. Studies have revealed that approximately 60% of men and 40% of women between the ages of 40 and 60 snore. Snoring among men is usually louder, and women tend to suffer more from this.

Snoring is not always harmless, however, because it frequently and virtually unnoticed leads to "sleep apnea", which is a condition in which the airways are temporarily closed off, resulting in a reduction of oxygen saturation in the blood. According to recent stu-

dies, about 14% of men and 7% of women suffer from these multiple breathing cessations.

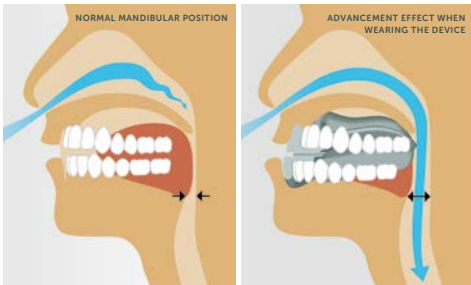
Snoring sounds are caused by fluttering movements of the soft palate, the base of the tongue and the soft tissues of the trachea. This can be caused by a slackening of the muscles during sleep, which is a tendency that increases with age. Similarly, obesity leads to a narrowing of the upper airway, as fatty tissue also accumulates in this area.

### **The snoring therapy system**

The Friedensstifter® works by advancing the lower jaw, which shifts the tongue in particular, but also the adjacent tissues, forward. This keeps the upper airway open and dilates it, which at the same time prevents snoring.

## Selection

The Friedensstifter® is selected individually after measuring the maximum advancement possibility. According to scientific findings, about 50 to 80% of the maximum advancement possibility is usually adequate to keep the upper airway open enough to allow trouble-free nighttime breathing – without snoring.

## **Fitting**

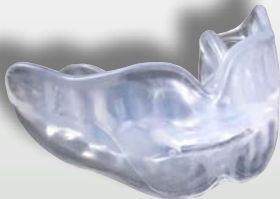
The device, which is made of thermoplastic material, is first heated and then adapted to the dentition of the upper and lower jaw.

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