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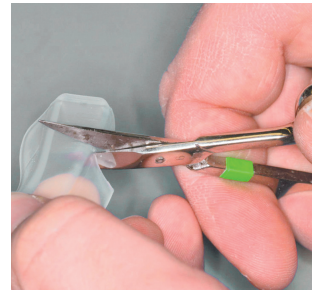
info@dr-hinz-dental.de | www.dr-hinz-dental.de

# BRUXI+

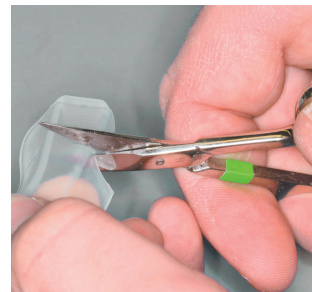
## Instructions for use for the production of a night guard

**BRUXI+ is an immediate help against teeth grinding and also helps 3- to 12-year-old children with mandibular joint problems.**

The contourless BRUXI+ resembles an impression tray and has the external appearance of a splint that can be made individually. For this purpose, an impression is made directly in the patient's mouth or with a jaw model using "DIRECT for BRUXI+" – a fast hardening silicone.

Bruxi+ is available in only one size for the lower jaw.

For use in milk teeth, cut about 1 cm of each side of the silicone splint with scissors. For this reason, a fine marking extending diagonally forward has been incorporated.

**Shortening the Bruxi+**

**1 Seal it with SILCOfix**

**2 Fill it with the DIRECT silicone**

**3 Impression  
(on the model or jaw)**

Before the „impression“ is made, the inner sides and edges of BRUXI+ have to be generously sealed several times with SILCOfix.

It hardens in the mouth after only about 90 seconds; if the jaw model is used, it takes about 3 minutes. After approx. 5 minutes, the excess material can be cut off with scissors and the edges smoothed out with sanding discs.

**Result of the impression**

**Further questions or problems? We will be happy to advise you.**

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