BRUXI+

D
GB
F
I
E

Patienteninformation

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Patient information

Teeth grinding in children

Approximately 20 % of the children and adolescents are affected by nighttime bruxism. Noisy bruxing can already occur after eruption of the primary molars at the age of two. Causes presumably are psychological reasons such as stress, anxiety, aggressiveness or an unfavorable occlusion that needs grinding-in.

Applied at an early stage – possibly from the age of three – **BRUXI+** can effectively stop bruxing when worn at nighttime for a few weeks. Also applicable for all other age groups.

How is **BRUXI+** worn?

BRUXI+ – a lower jaw silicone splint is only worn at nighttime and protects tooth enamel. The oral vestibule is automatically held in place by the lip muscles.

Customized **BRUXI+**

BRUXI+ rules

Your support and a few rules will help your child:

- 1 **BRUXI+** must be voluntarily worn.
- 2 Children who have not used a pacifier must first get used to having an extraneous body in the mouth. The "chewing gum effect" of the silicone base material helps this.
- 3 In the first week, for easier habituation, it should be worn one hour before going to bed and at night. If **BRUXI+** does not fall out of the mouth, nocturnal use is sufficient. If the silicone splint detaches from the teeth, wearing time should be extended (during the day by the hour while playing or watching TV).

- 4 As a rule, after wearing it regularly for about 8-12 weeks, **BRUXI+** can be taken off on a trial basis and the grinding behavior can be observed. If the damaging habit continues, wearing time is extended.
- 5 After successful treatment, **BRUXI+** should nonetheless still be worn for 6 to 8 additional weeks ("risk of relapse").
- 6 **BRUXI+** can be stored in the hygiene box.
- 7 Don't forget: Brush your teeth and **BRUXI+** on a regular basis!

If you have questions, please contact your dentist who will support the therapy!

Instructions for use

BRUXI+ can be shortened by the dentist if necessary. Clean **BRUXI+** thoroughly before using it for the first time under warm running water. Check **BRUXI+** regularly for places that are bitten through! If necessary, replace **BRUXI+**.

Cleaning instructions

Clean it thoroughly every day with a toothbrush under warm running water. Alternately, you can also use the OROFAN® dental cleaning tablets. **BRUXI+** is neither dishwasher-safe nor microwave-safe. Only clean the hygienic **BRUXI+** with a damp cloth.

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