OrthoP PreventAligner®

Kll-M hard

D
GB
F
I
E

Patienteninformation

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E

Patient information

Prevention-oriented early treatment

Harmful factors that have a lasting effect on facial growth and tooth positions require early **preventive treatment**, i.e. prophylactic treatment.

Impairments include harmful sucking habits, dysfunctions of the oral musculature - often associated with general postural weaknesses.

With the **OrthoPreventAligner®**-system, dysfunctions such as open mouth posture, incorrect tongue rest positions, faulty swallowing and a muscularly weak lip closure are corrected. These are often associated with the lower jaw distal bite (KII).

This also includes gapped and protruding incisors as well as frontally open bites, which result in speech disorders, restricted biting or chewing.

How jaw anomalies develop

There are congenital, inherited and acquired anomalies in the maxillofacial region.

For the most part, only the causes and effects of **“acquired”** tooth misalignments and jaw anomalies can be treated effectively using the prefabricated devices of the **OrthoPrevent Aligner®** system.

These anomalies are usually caused by non-physiological influences on the developing deciduous dentition, by sucking habits and muscular dysfunctions.

The tongue plays an essential role in the development and maintenance of dental anomalies: If the tongue is displaced from its resting position (the upper jaw) or if the tongue becomes lodged between the rows of teeth, the “acquired” jaw anomalies develop

For the treatment of faulty swallowing and speech errors, logopaedic therapy should also be provided.

The OrthoPreventAligner® KII-M

The **OrthoPreventAligner® KII-M** (KL II = lower jaw distal bite) is made of a highly elastic silicone. The lip shield for the upper jaw is extended and strongly tilted backwards. This completely encloses the incisors, which are usually tilted forward, and the alveolar extension.

The effect of the **OrthoPreventAligner® KII-M** to eliminate the incisor step is partly indirect: The stimulation of growth of the incorrectly positioned lower jaw leads to its regular position to the upper jaw and to the elimination of the incisor step.

In addition, there is a direct effect by resting the upper lip shield on the incisors, which usually protrude with gaps. This directs them towards the palate and thus contributes to the reduction of the incisor step.

Usually, the removal of the incisor step not only improves pronunciation, but also makes it easier to bite with the incisors.

KII-M Deciduous dentition

In the case of an open bite with
prolonged vestibular pelotte
(Similar to Fig.)

Development of lower jaw distal bite and an incisor step

If the incisors do not bite over each other like scissors and if there is a more or less large step between the teeth of the upper jaw and those of the lower jaw, the upper teeth **appear** to be too far forward.

In fact, however, the entire lower jaw is too far back.

Every person is born with a "baby's distal bite". For developmental reasons, most people correct this by themselves in the first year of life, however a quarter of them, they don't do this completely or not at all.

Irrespective of this, the upper incisors are often tilted further forward due to the insertion of the lower lip, thus additionally reinforcing the incisor step.

Since there is still general growth in children, the growth deficit can be compensated for by regularly wearing functional orthodontic appliances (including the **OrthoPreventAligner® KII-M**).

Recommended wear

In order to break harmful habits and to eliminate **mal-functions** as well as to classify "crooked" teeth, the **OrthoPreventAligner®** should be worn regularly for at least 2 hours a day and at night.

Wearing check

	Week 1		Week 2		Week 3	
	 hr.	 hr.	 hr.	 hr.	 hr.	 hr.
MON						
TUE						
WED						
THUR						
FRI						
SAT						
SUN						

Instructions for use

Rinse the **OrthoPreventAligner**[®] with warm water before using for the first time. The **OrthoPreventAligner**[®] can be disinfected briefly by boiling water after infections.

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, OROFAN[®] dental cleaning tablets can be used.

The **OrthoPreventAligner**[®] is not dishwasher or microwave safe. Only clean the hygienic **OrthoPrevent Aligner**[®] box with a damp cloth.

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